

2018 Circuit disruption 2

Permission to print:	Yes
Incident type	Good Catch Near Miss
Type of incident:	Equipment
Category	Circuit disruption
Description:	While priming the Maquet VKMO 31000 Oxygenator Lot number 92228467 Expiry 06-08-2019 it was noted that there was a fluid leak form the one way valve of the recirculation/deairing line that goes from the oxygenator/integrated ALF back up to the venous reservoir.
GOOD CATCH - what we	examination during priming picked up the leak
Preventive actions	The recirc line was replaced with an alternate sterile line and the rest of priming and the case were uneventful. The company was notified and has provided replacement stock of a different Lot number as well as extra recirc lines to use with remaining affected lot numbers on shelf
Manufacturer advised:	Yes
Discussed with team:	Yes
Ext Authority Advised	No
Hospital incident filed:	Yes
Knowledge issue	No
Rule issue	No
Skill issue	No